



FARO 'INSIDE' ENDOSCOPE

CLEANING & STERILISATION

1. Endoscope & Arms

- a. Clean the Endoscope and the button control panel using alcohol on a soft cloth or a specific disinfectant for hygienising surfaces

Never immerse the Endoscope in liquid solutions.

The Endoscope cannot be sterilised

- b. Take care not to scratch the surface of the optics glass lens. Remove dust with the air jet of a syringe.

2. Protective accessories

- a. Clean the accessories with alcohol on a soft cloth or with a specific disinfectant
 - i. take care not to allow the liquid to penetrate inside the accessory
 - ii. Take care not to scratch the glass lens
 - iii. Never immerse the accessories in a liquid solution
- b. If necessary, clean the air outlet nozzles with a probe
- c. To clean the glass lens of the accessories to remove any post-sterilisation haloes, use the supplied cleaning rod.
 - i. When the cotton cylinder of the cleaning rod is dirty or worn, unscrew it from the end section and replace it with a normal cotton wad (saliva roller)

3. The accessories can be sterilised in an autoclave with saturated aqueous vapour at 135°C Time in autoclave – approximately 3 minutes



TROUBLESHOOTING

Problem	Solution
Halo Stains	Clean the protective accessories and the Endoscope lens as specified above
Reflections	Make sure that the protective accessory is correctly secured
No Image	Check connections to monitor. Make sure you select the analogical input to the monitor
Disturbed video image	Check connections to the monitor. Make sure that there are no appliances generating disturbances in the vicinity of the Endoscope
Cleaning air not delivered or too little delivered	Clean nozzles as specified in paragraph 2b above
Endoscope 'warms' in the rear zones	Slight warming in the rear zones is normal during use. Do not leave the Endoscope switched on when not in use